# TRIGO

02-5041C

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**EN** This Instruction for use **MUST** be given to the user of the product.

**BEFORE** using this product, this Instruction for use **MUST** be read and saved for future reference.

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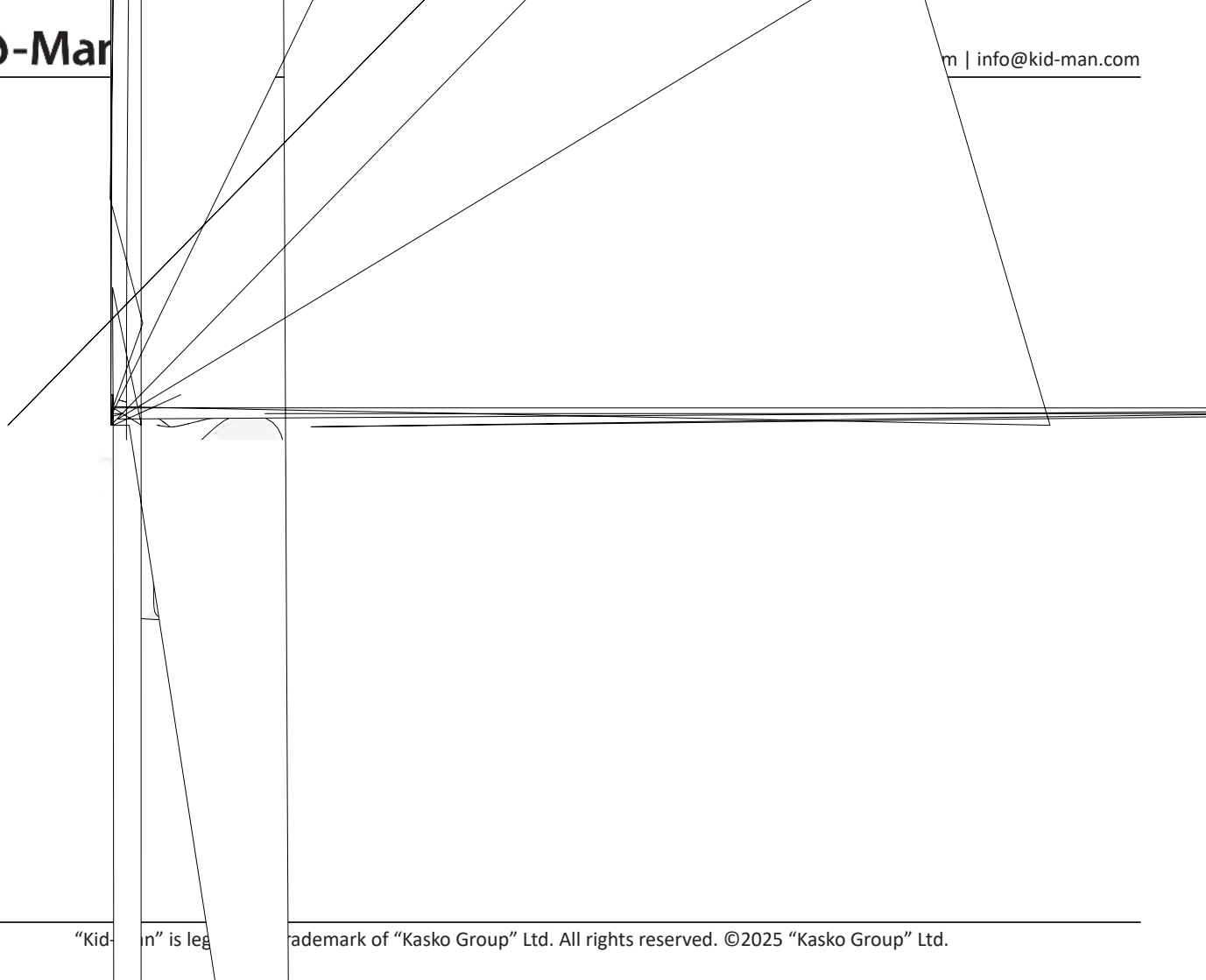
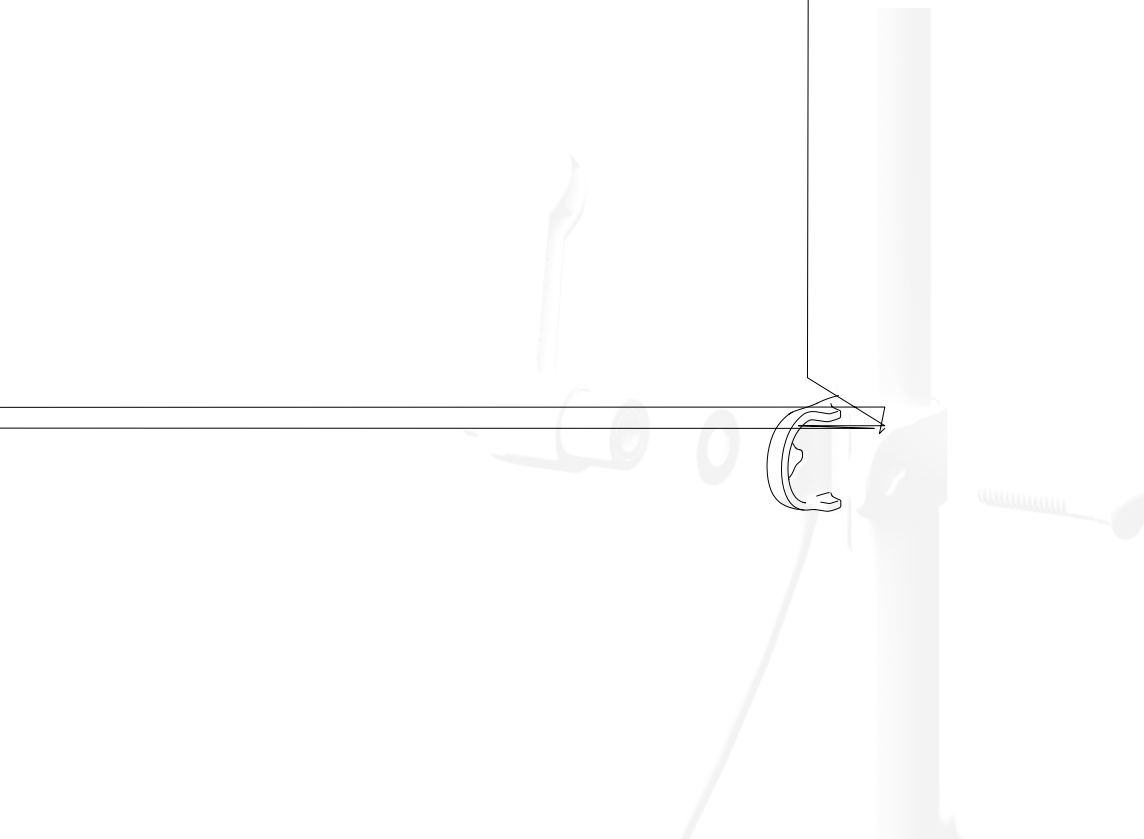
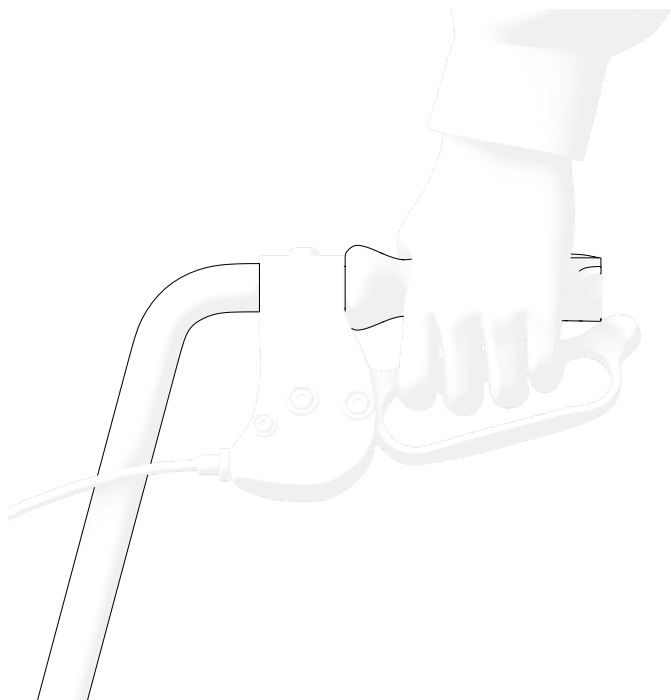
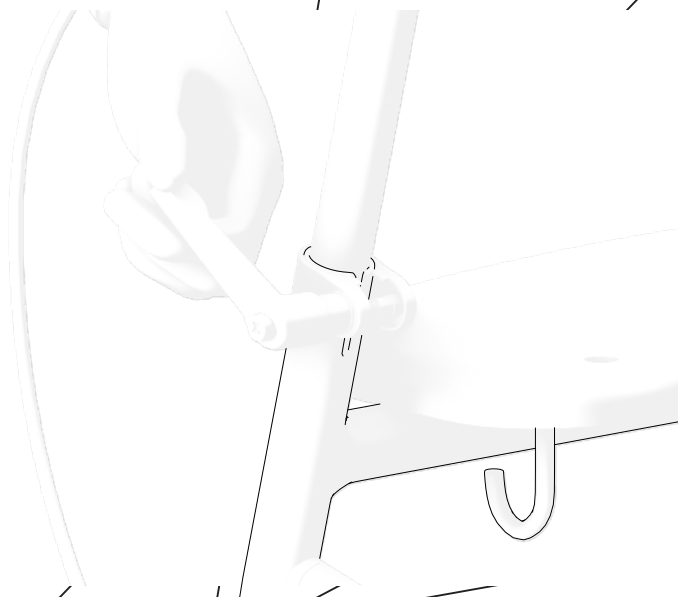
**FI**

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**SYMBOLS**

**In this Instruction for Use warnings are indicated by symbols.**

**The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.**

**i** Gives basic information, recommendations and information for efficient, trouble-free use.

**WARNING / CAUTION** - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

**ATTENTION!** This sign is used for very important parts concerning your safety. Please look at the operating instructions.

**SAFETY INFORMATION****WARNING**

The rollator is only suitable for single occupancy.

**CAUTION**

Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's

## SAFETY INFORMATION

### **WARNING**

The rollator is not intended for use by the blind.

### **WARNING**

Use slow speeds on gradients. Do not exceed the maximum gradient stated.  
Do not use escalators. Do not reverse down a gradient.

### **WARNING**

It is forbidden to exceed the maximum safe load of product.

### **WARNING**

While unpacking, be cautious to avoid catching your fingers between the wheel and the fork.

### **WARNING**

When using the wheeled rollator, use both brakes simultaneously to prevent the unbraked wheel from twisting and causing the rollator user to fall.

## GENERAL

### INTRODUCTION

This Instruction for use contains important information about the handling of the product. To ensure safety when using the product, read the Instruction for use carefully and follow the safety instructions.

3 wheel rollator is a walking device for indoor and outdoor use perfect for compensation and improvement respectively of restricted ability of walking.

### PREFACE

Thank you for the confidence shown in Kid-Man®. Every product is factory checked and left us in faultless condition.

We wish you much success and we hope UAB Kasko Group | Kid-Man® product will make a positive impact to your daily living.

### WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in an improper way. Warranty is only valid with fully filled warranty card. UAB “Kasko Group”, after receiving the claimed item from the purchaser within a period of 4 weeks, decides whether the issue is covered under warranty. If the issue is not covered under warranty and

## STANDARDS AND REGULATIONS

The 3 wheel rollator complies the requirements of:

- a) REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC.
- b) LST EN ISO 11199-2:2021 Assistive products for walking manipulated by both arms - Requirements and test methods - Part 2: Rollators (ISO 11199-2:2021) EN ISO 11199-2:2021.

## SERVICE LIFE

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this Instruction for Use.  
After then, rollator has to be recycled. Do not throw your rollator away with common waste (see chapter “DISPOSAL”).

## INFORMATION ABOUT MANUFACTURER

**Manufacturer:**

## INFORMATION ABOUT PRODUCT

**Product:** 3 WHEEL ROLLATOR

**Model:** TRIGO

**REF:** 02-5041C

## INTENDED PURPOSE AND INDICATIONS

3 wheel rollator is a walking device for indoor and outdoor use, perfect for compensation and improvement respectively of restricted ability of walking.

This 3 wheel rollator needs to fit to your specific health condition.

## INTENDED USERS

TRIGO 3 wheel rollator is intended to compensate and improve mobility of a person with restricted ability of walking with a maximum weight capacity of 120 kg.

### **WARNING**

#### **Risk of accidents.**

It is forbidden to exceed the maximum safe load of 3 wheel rollator.

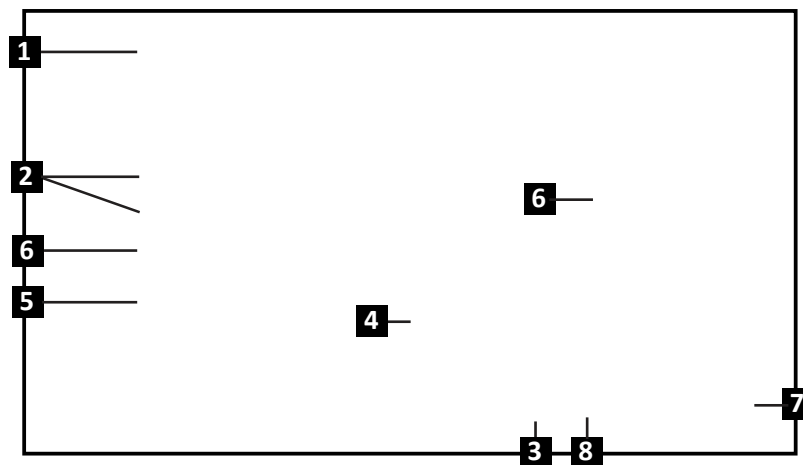
## CONTRA-INDICATIONS

There are no contraindications known, when using the TRIGO as intended.

**PRODUCT LABEL**

The identification label is attached to the product (see picture).

1. Product
2. Model
3. Maximum load
4. Manufacturer / contact details
5. Date of manufacture
6. UDI / SN code
7. CE mark
8. Consult instructions for use

## COMPONENTS AND FUNCTIONS

## SETTING UP THE ROLLATOR

**To unfold** the rollator:

1. Ensure the rollator is on solid even ground;
2. Hold the rear frame and push them outward to expand the rollator frame (Picture 1);
3. Ensure all parts are securely locked before use.

**To fold** the rollator:

1. Locate the folding red lever near the middle of the frame (Picture 2);
2. Lift the lever to release the locking mechanism;
3. Gently push the sides inward to fold the rollator (Picture 1);
4. Secure the folded position if needed and store.

### **WARNING**

Danger of trapping your finger!

**Setting up** the height adjustment lever:

1. Align the bolt with the height adjustment bracket and insert it through;
2. Place the washer, attach the height adjustment lever, and tighten it securely (Picture 3).

## OPERATION

### **Braking**

The efficiency of the brakes can be affected by abrasion of the wheels. Therefore, test the brakes regularly. For desired braking

## OPERATION

### **To operate the brakes**

To activate the brakes, squeeze the brake levers up and hold them there (as indicated by the arrow (Picture 4)). To release the brakes, release the levers.

To engage parking brakes of the 3 wheel rollator, depress the brake levers until it locks and remains in the down position, as shown by the arrow (Picture 5).

### **Adjusting the height of the handles**

Adjust the plastic handles on the left and right sides of the rollator to the desired height (Picture 6), then lock the lever in place (Picture 7).

### **Walking**

When you walk with your rollator make sure you walk upright, with the rollator near your body. It will give you the correct body posture (Picture 8).

### **Sitting on a rollator**

The rollator is not designed for sitting. Do not sit on the rollator, including on the 3-wheel rollator basket (Picture 10). Using the rollator in this way can result in damage or personal injury.

## MAINTENANCE

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person. Below You will find recommended maintenance's schedule.

|                                                       | Before use | Once a week | Once a month | Once a year |
|-------------------------------------------------------|------------|-------------|--------------|-------------|
| Check that the casters are seated correctly           |            | X           |              |             |
| Visual check                                          |            | X           |              |             |
| Check the folding mechanism                           |            |             | X            |             |
| Check the brakes                                      | X          |             |              |             |
| Clean the casters                                     |            |             | X            |             |
| Have 3 wheel rollator checked by an authorized dealer |            |             |              | X           |

### WARNING

Repairs, cleaning, or adjustments should be performed by an authorized service person while the user is not using the rollator.

### CAUTION

## CLEANING AND DISINFECTION

Should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent.

- i** Sand and seawater can damage the bearings and steel parts of rollator can rust if the surface is damaged. If the product is exposed by sand and seawater for short periods, clean it after every trip to the beach.

## STORAGE

The product should be stored in a dry environment, away from direct sunlight.

### Environmental conditions

Keep TRIGO under these conditions:

- Temperature limitation -20° C to +40° C;
- 

### WARNING

The rollator must not be permanently exposed to temperatures greater than +40°C. Otherwise the frame as well as seat, backrest and armrests can heat up, which might cause burns.

### WARNING

Never store other items on top of the product.

- i** After long-term storage (more than 3 months) the product must be inspected (see chapter “MAINTENANCE”).

### RE-USE

The 3 wheel rollator is suitable for re-use. The following actions should be taken: Cleaning and disinfection (see chapter “CLEANING AND DISINFECTION”). Inspection (see chapter “MAINTENANCE”).

TRIGO should be adapted for the new user in accordance with the service documentation.

### DISPOSAL

Be environmentally aware and dispose of your product properly. Disposal is subject to national and local regulations. To have the 3 wheel rollator properly recycled, please contact your local community, disposal facility or our service department.

### SPECIFICATIONS

| Model                     | TRIGO    |
|---------------------------|----------|
| REF                       | 02-5041C |
| Maximum safe load (kg)    | 120      |
| Maximum load of tray (kg) |          |
|                           |          |
|                           |          |
|                           |          |
|                           |          |
|                           |          |
|                           |          |
|                           |          |

All materials used are protected against corrosion. We use only REACH compliant materials and component.

## INCIDENTS

**Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.**

### CONTACTS:

**Manufacturer:** UAB “Kasko Group” (registered trade mark Kid-Man®)

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