# HUGO

01-3550

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FI

Instruction for use

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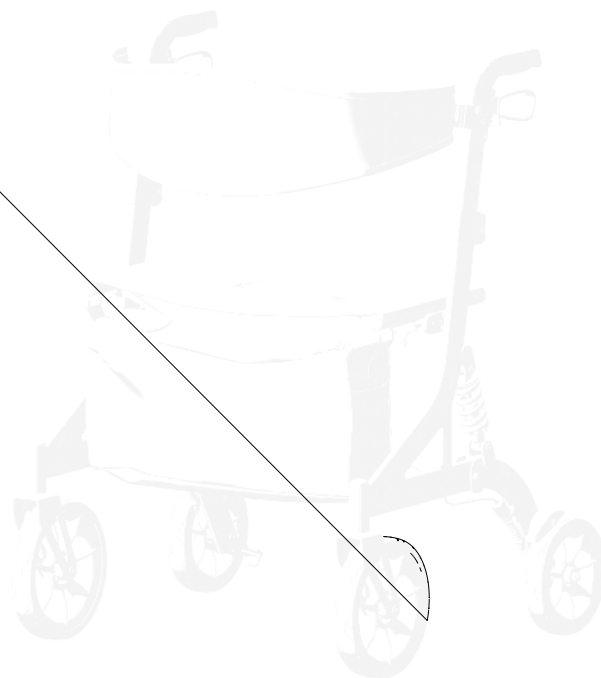
**EN** This Instruction for use **MUST** be given to the user of the product. **BEFORE** using this product, this Instruction for use **MUST** be read and saved for future reference

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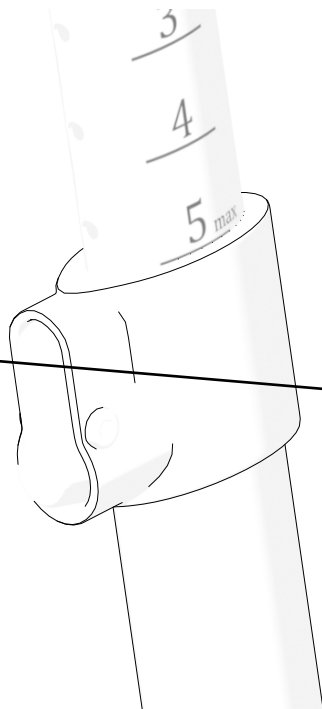
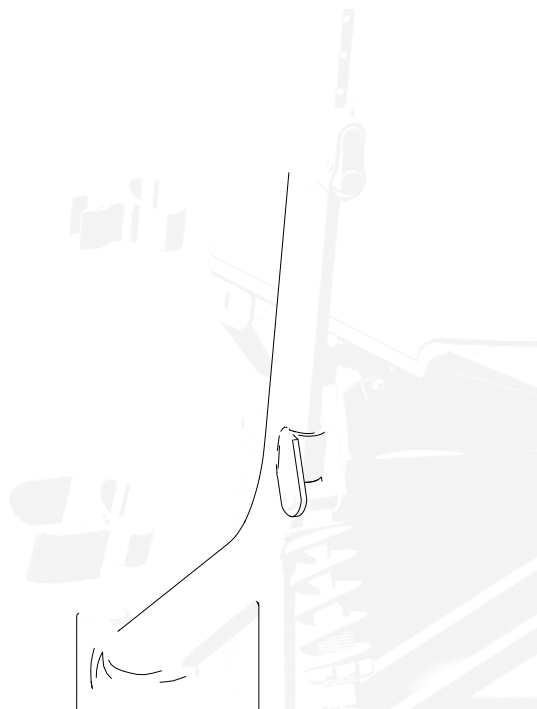
gur 10 /

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**EN****SYMBOLS**

In this Instruction for Use, warnings are indicated by symbols. The warning symbols are accompanied by a heading

**WARNING / CAUTION** - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

**ATTENTION!** This sign is used for very important parts concerning your safety. Please look at the operating instructions.

**SAFETY INFORMATION****WARNING**

The rollator is only suitable for single occupancy.

**WARNING**

Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items further than your arm will extend.

**WARNING**

Do not attempt to ride over curbs or obstacles. Doing so may cause your rollator to tip over and cause bodily harm to you or damage to the rollator.

**WARNING**

It is forbidden to smoke or to be close to the flames, using a rollator.

#### **WARNING**

The rollator is not intended for use by the blind.

#### **WARNING**

**Risk of burning.** The rollator must not be permanently exposed to temperatures greater than +40 °C. Otherwise, the frame as well as the armrests can heat up, which might cause burns.

#### **CAUTION**

**Risk of getting fingers caught.** There is always a risk of getting, e.g., fingers or arms, caught in the moving parts of the rollator.

#### **WARNING**

**Risk of accidents.** Safety devices that are incorrectly set or no longer working (e.g., brakes) can cause accidents. Always check that the safety devices are working before you use the 4 wheel rollator, and have them checked regularly by the authorized dealer.

#### **WARNING**

**Risk of accidents.** For proper and safe use, we recommend you follow the safety instructions while using your rollator. HUGO, was designed to ensure stability while walking. Every movement on the rollator affects the center of gravity, which may cause the device to tip over.

#### **WARNING**

Use slow speeds on gradients. Do not exceed the maximum gradient stated. Do not use escalators. Do not reverse down a gradient.

#### **WARNING**

It is forbidden to exceed the maximum safe load of product.

#### **WARNING**

## GENERAL

### INTRODUCTION

This Instruction for Use contains important information about handling the product. To ensure safety when using the product, read this Instruction for Use carefully and follow the safety instructions.

### PREFACE

Thank you for the confidence shown in Kid-Man®. Every product is factory checked and left us in faultless condition. We wish you much success and we hope UAB Kasko Group | Kid-Man® product will make a positive impact to your daily living.

### WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB Kasko Group | Kid-Man® (hereinafter called Manufacturer) after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, Manufacturer, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

## STANDARDS AND REGULATIONS

The 4 wheel rollator complies with the requirements of:

- a) REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC.
- b) LST EN ISO 11199-2:2021 Abiem rankomis valdomos pagalbinės vaikščiojimo priemonės. Reikalavimai ir bandymo metodai. 2 dalis. Vaikštynės su ratukais (ISO 11199-2:2021) Assistive products for walking manipulated by both arms - Requirements and test methods - Part 2: Rollators (ISO 11199-2:2021) EN ISO 11199-2:2021.

## SERVICE LIFE

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this Instruction for Use.

After then, the device must be recycled. Do not throw your product away with common waste (see chapter “DISPOSAL”).

## INFORMATION ABOUT MANUFACTURER

### INTENDED PURPOSE AND INDICATIONS

4 wheel rollator HUGO is a walking device for indoor and outdoor use, perfect for compensation and improvement respectively of restricted ability of walking.

This 4 wheel rollator needs to fit to your specific health condition.

### CONTRA-INDICATIONS

There are no known contraindications when using the 4 wheel rollator HUGO as intended.

### INTENDED USERS

4 wheel rollator HUGO is intended to compensate and improve mobility of a person with restricted ability of walking with a maximum weight capacity of 205 kg.

#### **WARNING**

Risk of accidents. It is forbidden to exceed the maximum safe load of the 4 wheel rollator.

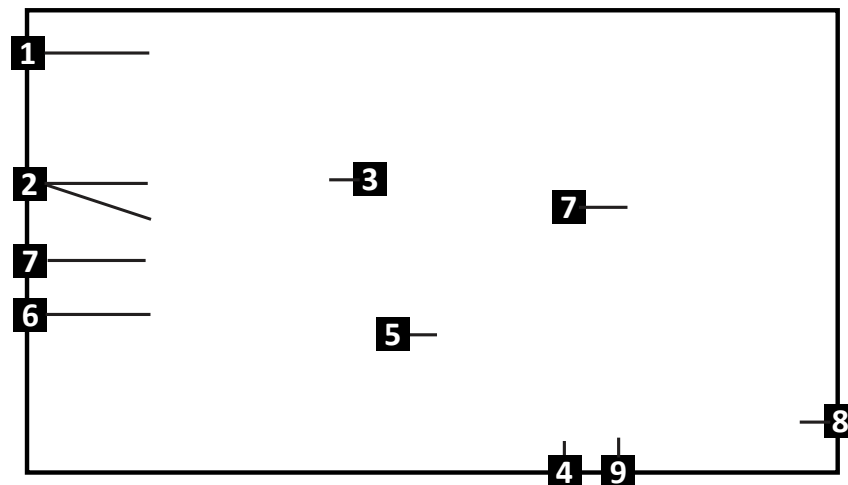
#### **CAUTION**

The 4 wheel rollator is appropriate for use by others. In order to do this, the rollator must first be thoroughly cleaned and disinfected. Then, the all components of the rollator must be get checked by an authorised service dealer for general condition, wear and shall get repaired if necessary.

## PRODUCT LABEL

**The identification label is attached to the frame of your product (see picture).**

1. Product
2. Model
3. Size (seat width)
4. Maximum load
5. Manufacturer / contact details
6. Date of manufacture
7. UDI / SN code
8. CE mark
9. Consult instructions for use

### CONTENT OF PACKAGE

The 4 wheel rollator HUGO is delivered in a cardboard packaging and are ready for use. After unpacking, please keep the cardboard packaging if possible. You may need it later for storage or for the return of the device. Once parcel is received, please check whether it is not damaged during transportation.

- i** If you see signs of damage, please make a note of it to the transport company at the same time.
- i** Please check the contents of the package, and if any of the parts listed below are missing, contact your local dealer.

#### Content of package:

1. 1 x Backrest
2. 1 x Folded rollator
3. 1 x Bag
4. 1 x Walking stick holder set
5. Instruction for use

## COMPONENTS AND FUNCTIONS

### Overview

1. Backrest
2. Seat
3. Bag
4. Front wheel
5. Handle
6. Brake
7. Height adjustable button for handles
8. Frame
9. Brake wire
10. Rear wheel

## SETTING UP THE ROLLATOR

**To unfold** the rollator:

1. Ensure the rollator is on solid, even ground;
2. With your hands flat, press the seat frame down until the seat surface flattens out and is fully set between the frame (Pictures 1 - 2);
3. Attach the backrest to the designated mounting points on the rollator (Picture 4);
4. Hang the textile bag onto the two bag holders at the front of the rollator (Picture 5);
5. Attach the walking stick holder to the frame as shown in Pictures 8 - 9, if necessary.

**To fold** the rollator:

1. Locate the folding strap on the seat (Picture 3);
2. Raise the seat using the lifting strap to bring the rollator upward. As you lift, the rollator will automatically fold together (Picture 1);
3. Detach the walking stick holder before folding to avoid potential damage;
4. Secure the folded position if necessary and store the rollator.

### **WARNING**

Danger of trapping your finger!

## OPERATION

### **WARNING**

Before settling on your seat, make sure the brakes are engaged!

## OPERATION

### **Adjusting the height of the handles**

While pressing the height adjustment button with one hand, use the other hand to raise or lower the handle to the desired height (Picture 6). Repeat the same for both handles on the left and right sides. The height must be the same on both sides.

The height can be adjusted to 5 different positions. The positions are marked on the inner side of the handles (Picture 7).

### **Walking**

When you walk with your rollator make sure you walk upright, with the rollator near your body. It will give you the correct body posture (Picture 11).

### **Sitting on the rollator**

Make sure the rollator is secured with the parking brakes before sitting on the seat.

When seated, always place your feet between the rear wheels (Picture 12).

### **Moving onto the sidewalk**

We recommend going up onto the sidewalk with the help of an assistant. Steps for the assistant:

- Position the rollator straight, directly in front of the curb.
- Tilt the rollator backward by pressing down on the tipping aid with your foot and slowly move forward until the front wheels roll over the curb (Picture 10).
- Push the rollator straight forward until the rear wheels touch the curb.
- Lift the rollator slightly and push it forward until the rear wheels are on the sidewalk.

**MAINTENANCE**

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person. Below You will find recommended maintenance's schedule

|                                                       | Before use | Once a week | Once a month | Once a year |
|-------------------------------------------------------|------------|-------------|--------------|-------------|
| Check that the wheels are seated correctly            |            | X           |              |             |
| Visual check                                          |            | X           |              |             |
| Check the folding mechanism                           |            |             | X            |             |
| Check the brakes                                      | X          |             |              |             |
| Clean the wheels                                      |            |             | X            |             |
| Have 4 wheel rollator checked by an authorized dealer |            |             |              | X           |

**WARNING**

Repair, cleaning, or adjustment of the product must be performed by an authorized service person while the product is not in use.

**CLEANING AND DISINFECTION**

## STORAGE

The product should be stored in a dry environment, away from direct sunlight. It should always be kept clean and free of dust.

### WARNING

The product must not be permanently exposed to temperatures greater than +40 °C. Otherwise, the frame and materials may overheat, which could cause burns or material damage.

### WARNING

Never place or store other items on top of the product.

After long-term storage (more than 3 months), the product must be inspected before use (see chapter “MAINTENANCE”).

## ENVIRONMENTAL CONDITIONS

Keep the product under these conditions:

Temperature limitation -20°C to +40°C;

Relative humidity 20% to 75%;

Atmospheric pressure 800 hPa to 1060 hPa.

## RE-USE

The product is suitable for re-use. The following actions must be carried out:

Cleaning and disinfection (see chapter CLEANING AND DISINFECTION); Inspection (see chapter MAINTENANCE);

The product should be adapted for the new user in accordance with the service documentation.

## DISPOSAL

**SPECIFICATION**

| <b>Model</b>                      | <b>HUGO</b> |
|-----------------------------------|-------------|
| <b>REF</b>                        | 01-3550     |
| <b>Maximum safe load (kg)</b>     | 205         |
| <b>Weight (kg)</b>                | 9.5         |
| <b>Overall length (cm)</b>        | 72          |
| <b>Overall width (cm)</b>         | 77          |
| <b>Overall height (cm)</b>        | 90 - 99     |
| <b>Seat height (cm)</b>           | 63          |
| <b>Basket length (cm)</b>         | 14          |
| <b>Basket width (cm)</b>          | 52          |
| <b>Basket depth (cm)</b>          | 24.5        |
| <b>Width between armrest (cm)</b> | 61          |
| <b>Front wheel size (cm)</b>      | 23.5 x 4.5  |
| <b>Back wheel size (cm)</b>       | 20 x 3.5    |

## INCIDENTS

**Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.**

### CONTACTS:

**Manufacturer:** UAB “Kasko Group” (registered trade mark Kid-Man®)

**Address:** A.Vivulskio 12D, Vilnius, LT-03221, Lithuania

**Phone:** +370 5 265 0000

**E-mail:** [info@kid-man.com](mailto:info@kid-man.com)

**Website:** [www.kid-man.com](http://www.kid-man.com)

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