LELO

LUNA Smart Bead™

USER MANUAL & WARRANTY INFORMATION

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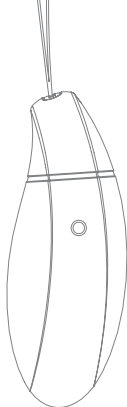
Congratulations on acquiring the LUNA Smart Bead™! We know you're going to love it, so before you start getting more closely acquainted with your new Personal Pleasure Trainer, please take a moment to look through this user manual.

REGISTER WARRANTY

To register your warranty, find your warranty number on the LELO Authenticity card provided. Then visit **LELO.com** and click on REGISTER in the top right side of the page. This allows you to confirm the authenticity of your purchase and protect your investment.

CONTENTS

OVERVIEW	05
WARRANTY TERMS & CONDITIONS	06
LUNA SMART BEAD™ PRODUCT DIAGRAM	07
OPERATING INSTRUCTIONS	08
CLEANING, STORAGE AND SAFETY	13
SPECIFICATIONS	14
DISCLAIMER	15

OVERVIEW

The LUNA Smart Bead™ - is an intelligent pleasure product that helps you to enjoy longer, stronger and more regular orgasms over time. The LUNA Smart Bead™ can also be used as a vibrating massager that relaxes your body and heightens the mood.

As you use the LUNA Smart Bead™ over weeks and months, it automatically monitors your orgasm-potential and adjusts the frequency of vibrations to match your performance-level as it improves over time.

Always consult your doctor before using LUNA Smart Bead™ during pregnancy.

WARRANTY TERMS & CONDITIONS

1-YEAR WARRANTY

LELO warrants this product for a period of ONE (1) YEAR, after the date of original purchase, against defects due to faulty workmanship or materials. If you discover a defect and notify LELO during the warranty period, LELO will, at its discretion, replace the product free of charge.

The warranty covers working parts that affect the function of the product. It does NOT cover cosmetic deterioration caused by fair wear and tear or damage caused by accident, misuse or neglect. Any attempt to open or take apart the product (or its accessories) will void the warranty.

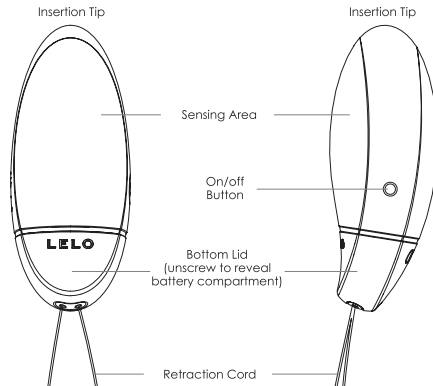
Claims under warranty must be supported by reasonable evidence that the date of the claim is within the warranty period. To validate your warranty, please keep your original purchase receipt together with these warranty conditions for the duration of the warranty period.

To claim your warranty, you MUST register your product at [LELO.com](https://www.lelo.com) and follow the instructions to make your claim. You will be sent a Return Merchandise Authorization (RMA) number which should be returned with the LELO product to your nearest LELO office carriage paid. Shipping costs are non-refundable. This undertaking is in addition to consumer's statutory rights and does not affect those rights in any way.

10-YEAR QUALITY GUARANTEE

As a continuation of the terms of LELO's full one (1) year warranty, LELO's ten (10) year quality guarantee makes owners eligible to purchase a new product from [LELO.com](https://www.lelo.com) for 50% off the listed price. See the CUSTOMER CARE section of [LELO.com](https://www.lelo.com) for Terms & Conditions.

LUNA SMART BEAD™ PRODUCT DIAGRAM

OPERATING INSTRUCTIONS

TURNING ON/OFF

- To turn on your LUNA Smart Bead™, press the **O** button.
- The device will turn off automatically after an exercising routine is complete, but should you need to turn it off before completion of your routine, press and hold the **O** button for 2 seconds.
- As an added option, this device also provides a continuous vibration mode purely for pleasure. Press and hold the **O** button for 3 seconds before or after your routine.

CHANGING THE BATTERY

- One fully charged AAA-battery should last you for up to 200 routines.
- If the device fails to turn on, or there is a noticeable drop in vibration intensity, it is time to change the battery.
- Don't worry about losing your recorded training history, the LUNA Smart Bead™ will remember your progress even after a battery change.

BEFORE FIRST USE

1. **Unscrew the bottom lid of the LUNA Smart Bead™ and insert the AAA battery.**
2. **Screw the lid back on and make sure it fits snugly. With the lid properly fastened, the LUNA Smart Bead™ is 100% waterproof. Always wash thoroughly before and after each use.**
3. **Please do NOT place your LELO directly in a pan of boiling water. We strongly recommend LELO's purpose-made Cleaning Spray or other cleaners specifically designed for the purpose.**

GETTING STARTED

- Press the **O** button to turn on the device. The light on the LUNA Smart Bead will flash to indicate which level you are in. Your starting level is set to 3 for your first routine. The routine will start automatically after 30 seconds, giving you time to insert.
- Find a comfortable position for insertion. Many women find lying on your back or standing most comfortable. For easy insertion, apply a bit of water-based lubricant to the tip of the LUNA Smart Bead™.
- Insert the device as you would a tampon, with the **O** button on the left-hand side, leaving the extraction cord outside the body for safety and easy removal.
- Once comfortably inserted, 3 quick vibrations signal the start of the routine.
- Now simply squeeze when you feel the vibrations, and relax when they stop. After first use, your routine will be adjusted automatically according to your performance.
- You will know when the standard routine is completed when you feel a final series of 3 quick vibrations, and the LUNA Smart Bead™ turns off by itself. The next time you turn it on, it will flash between 1-5 times to show what level you are in, and remember this for next use.

USING THE LUNA SMART BEAD™

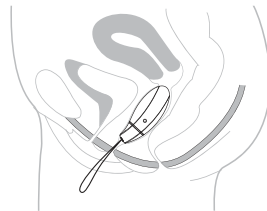
- Just like a personal trainer, the LUNA Smart Bead™ will set you a routine that is simple to follow, safe to use and completely tailored to your individual needs.
- All you have to do is SQUEEZE when it vibrates and REST when it stops.
 - When a vibration is felt, squeeze and hold for the duration of the vibration. The vibration intensity will increase as you squeeze, which you may or may not feel - the purpose of this is to ensure you are being properly guided in your routine.
 - Once the vibration stops, release, relax and wait for the next vibration to start.
 - Repeat this throughout the routine.
- Repeat this until quick vibrations will signal you have completed your routine, and turn off automatically.

IMPORTANT

- **Do not use anally.**
- **Do not twist device when inserted, always pull using the retraction leash.**

AFTER THE ROUTINE

- Remove the LUNA Smart Bead™ by relaxing your muscles, gripping the retraction string and carefully pulling out the device.
- Before storing for next use, make sure the LUNA Smart Bead™ is completely clean and dry.

PROGRESSING TO THE NEXT LEVEL

The LUNA Smart Bead™ has an integrated pleasure-training feature that will automatically remember your last routine-level and choose a suitable program for you based on your body.

As you advance through the 5 built-in routines, your pleasure will be enhanced through greater orgasm control. To track your progress, the LED located by the **O** button will flash when you turn on the LUNA Smart Bead™ to indicate your current level: one flash for level 1, five flashes for level 5.

Initial use of the LUNA Smart Bead™ will begin at Level 3 and adjust from that point depending on your performance.

Simply follow the vibration-guided routine of 'SQUEEZE & RELAX' and you can look forward to enhanced orgasms in no time.

LEVEL 1	LEVEL 2	LEVEL 3	LEVEL 4	LEVEL 5
Beginner	Amateur	Intermediate	Advanced	Master
For the uninitiated who has never practiced before → For the Master with the ultimate orgasm potential				

CLEANING, STORAGE AND SAFETY

Your LELO product's lifespan will be greatly increased with proper maintenance and care.

LELO recommends that you use a water-based solution such as LELO's Personal Moisturizer. Avoid using a silicone-based lubricant as this could make the silicone permanently tacky. If you wish to use a silicone-based lubricant, perform a patch test first.

Never use massage oil or hand cream as lubricants with your LELO. Always clean your LELO thoroughly before and after each use. The most effective way of doing this is to use a purpose-made cleaning solution such as LELO's purpose-made Cleaning Spray. Alternatively, you can wash the silicone with warm water and antibacterial soap, rinse with clean hot water and pat dry with a lint-free cloth or towel. Never use cleansers containing alcohol, petrol or acetone.

Your LELO is 100% waterproof and submersible to a depth of 1 meter.

Avoid leaving your LELO in direct sunlight and NEVER expose it to extreme heat. Store your LELO in a dust-free place and keep it away from toys of other materials.

IMPORTANT: For Adult Use Only. Those who may be pregnant, have a pacemaker, diabetes, phlebitis or thrombosis should consult with a medical professional regarding the risk of blood clots before using any personal massage product.

SPECIFICATIONS

MATERIALS:	Body safe PC ABS / silicone
FINISH:	Matte
SIZE:	1.3 x 1.2 x 3.3in
WEIGHT:	50g
BATTERY:	Standard AAA
STANDBY TIME:	Up to 30 days
FREQUENCY:	< 200Hz
MAXIMUM SOUND LEVEL (VOLUME):	< 50dB
INTERFACE:	1-button interface with biofeedback-enabled Personal Pleasure Training functions.

Disposal of old electronic equipment (applicable in the EU and other European countries with separate waste collection systems).

The crossed-out dustbin symbol indicates that this product should not be treated as household waste, but rather be brought to the appropriate collection point for recycling of electrical and electronic equipment.

DISCLAIMER

Users of this product do so at their own risk. Neither LELO nor its retailers assume any responsibility or liability for the use of this pleasure object. Further, LELO reserves the right to revise this publication and to make changes from time to time in the contents hereof without obligation to notify any person of such revision or changes.

Model may be changed for improvements without notice.

© 2014/2015 LELOi AB. ALL RIGHTS RESERVED.

DESIGNED AND DEVELOPED BY LELO SWEDEN.

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